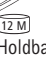
# Linovera® Scar Repair

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## EN INSTRUCTIONS FOR USE

### 1. DESCRIPTION

Linovera® Scar Repair has an innovative and exclusive formulation of Hyaluronic acid, Asian centella and Syricalm® which is a combination of natural extracts. The prolonged use of Linovera® Scar Repair helps the processes of elasticity improvement and natural colour of the healed tissue.

### 2. INSTRUCTIONS FOR USE

Apply the gel twice a day, preferably in the morning and at night.

Spread the gel over clean and dry skin, using the fingers to softly massage the scar.

Gel absorption is quick.

The first signs of improvement can be seen after 8 weeks of use. Best results can be seen between 2 to 4 months of continuous use.

It is recommended to use a soft silicone bandage (for example Askina® Scar Repair) to protect the skin from UV rays and improve the effect of the product.

Hypoallergenic product. Tested under dermatological control.

### 3. PRECAUTIONS OF USE

Do not apply Linovera® Scar Repair over damaged or not completely healed wounds.

### 4. CONTRAINDICATIONS

Do not use Linovera® Scar Repair in case of a known sensitivity to any of its components.

### 5. STORAGE

Store Linovera® Scar Repair at room temperature, away from direct sunlight and out of humidity.

### 6. PACKAGING

Linovera® Scar Repair is presented in a 15 g container.

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